


Complete Curtain Change in 30 Seconds Without a Ladder

NEW IN 2020
REUSABLE CURTAINS

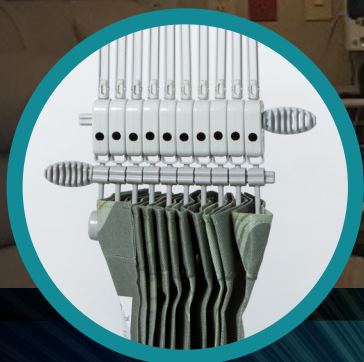

Rapid Refresh

Reusable and Disposable Curtains

Speed. Convenience. Infection Prevention.

TT TEAM
Medical
A DIVISION OF TEAMTechnologies

 **Rapid Refresh™**
SOLUTIONS

Rapid Refresh™ | Privacy Curtains

Change curtains in just 30 seconds and increase compliance while leading to an overall safer environment for the healthcare workers and their patients.

Why Rapid Refresh Privacy Curtains?

- Curtains can be changed in 30 seconds
- Time Strip Label and Curtain App make managing your curtain changes quick and easy
- No Mesh
- Eliminates the need for ladders
- Compatible with existing track and hooks
- Panels easily snap together for complete coverage and privacy
- Rapid Refresh Disposable and Reusable Curtains color coordinate
- Proven to be ergonomically better for healthcare employees
- Elimination of cleaning/laundry and transportation costs with disposables
- Turn rooms and bays over faster
- Multiple rod lengths for uniform look from a 7' ceiling to a 12' ceiling
- Reduction in curtain SKUs to one
- NFPA 701 Tested





Rapid Refresh™ | Shower Curtains


Our shower curtains support infection prevention protocol and are easy to remove and replace with no need for ladders or special tools.

Shower Curtains


- Shower Curtains can hang on shower rods or standard curtain track
- Can be cut to custom fit to length of your space
- Colors are complementary to the Rapid Refresh Privacy Curtain Line

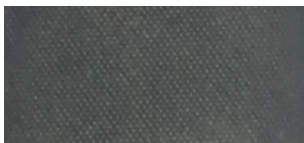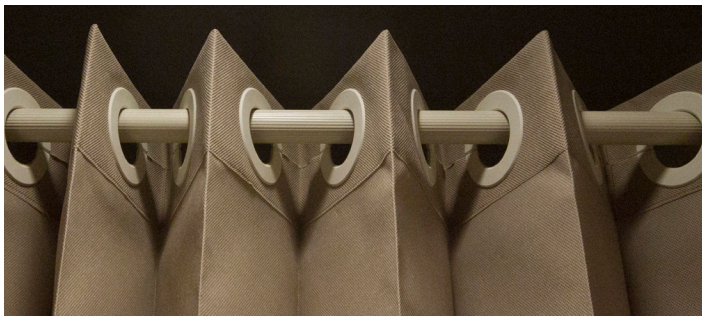

RR-6400DG (Dark Grey)

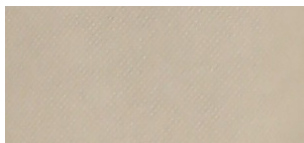

RR-6400G (Light Grey)

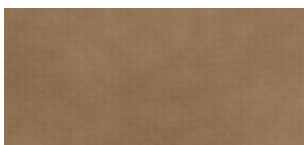

RR-6400T (Tan)




Rapid Refresh™ | Privacy Curtains

Designed to help EVS gain back time by quickly changing curtains and documenting it for your protocol.

ICP Medical Timestrip® Technology

- Easy to see date of curtain installation
- Helps monitor how long a curtain has been hanging from 1 to 6 months
- The indicator, when completely red, visually alerts healthcare workers to change the curtain
- Timestrip® technology available on disposable curtains
- Bar Codes available on all curtains

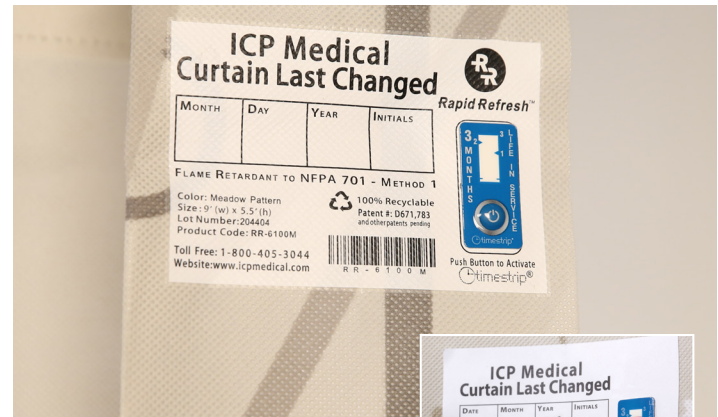


The ICP Medical Curtain Change Management System (CCMS) App

- Easy-to-use step-by-step process on Technician mobile device
- Real-time reporting
- Easy to manage curtain change policy by building or department
- Easy-to-read Dashboard for managers to quickly review work-flow
- RFID chip and a barcode on the label make it easy to scan and track the curtain. RFID chip available only on reusable curtains
- Available at the App Store and Google Play

The result:

- Makes tracking of curtain changes easier
- Quick way to identify curtains in or out of compliance with hospital protocol

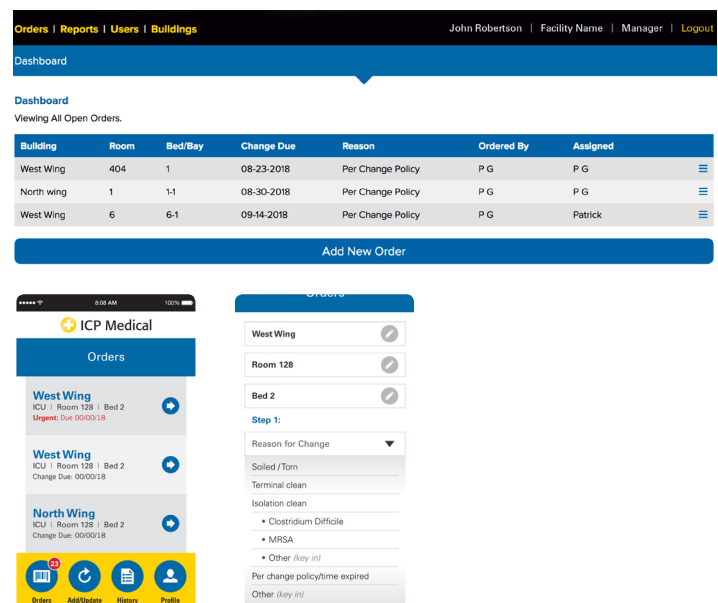

Rapid Refresh™ | Privacy Curtains

Rapid Refresh™ Privacy Curtains are as aesthetic as they are functional. Designed to fit any decor, choose from these standard colors and patterns to complement your environment.




Fall Pattern Curtain
RR-6200FALL
RR-6500Fall - Reusable



Meadow Pattern Curtain
RR-6200M
RR-6500M - Reusable



Tan Meadow Pattern Curtain
RR-6200TM
RR-6500TM - Reusable

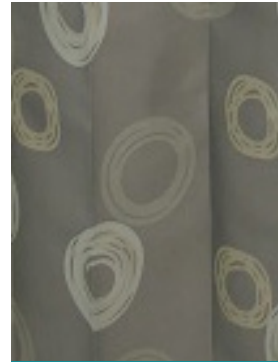

Circles Pattern Curtain
RR-6500CIRCLES
RR-6500Cirles - Reusable



Teal Flower Pattern Curtain
RR-6200TF




Beige Pattern Curtain
RR-6200BP



Tan Stripe
RR-6200ST



Light Blue Curtain
RR-6200LB

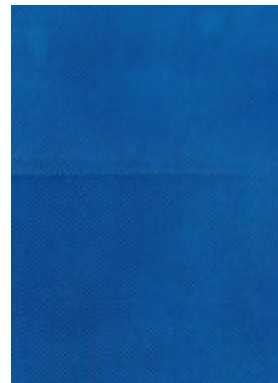

Dark Blue Curtain
RR-6200DB



Tan Curtain
RR-6200T



Fern Pattern Curtain
RR-6200FERN




Child's Play Pattern Curtain
RR-6200CP




Rainbow Pattern Curtain
RR-6200RB




Zoo Animals Pattern Curtain
RR-6200ZA

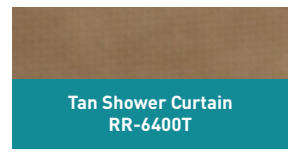

Tan Shower Curtain
RR-6400T

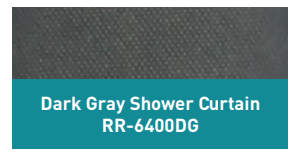

Dark Gray Shower Curtain
RR-6400DG

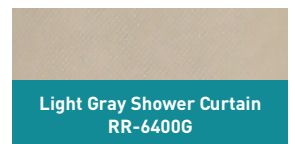

Light Gray Shower Curtain
RR-6400G

Rapid Refresh™ | Privacy Curtains

Rapid Refresh™ Curtains can be changed in as little as 30 seconds. The patented release mechanism allows the curtain to be changed without the need for a ladder reducing the threat of falls, increasing the speed of a room cleaning and increasing compliance.



1 Open new curtain package at perforation.



2 Remove loose key from package.




3 Gather old curtain and insert key through upper holes.



4 Turn key toward you to release curtain.



5 Discard old curtain.

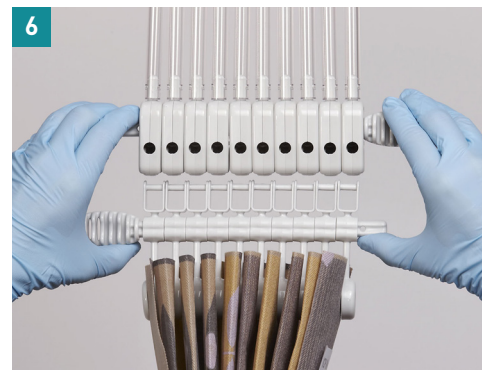

6 Line up new curtain with rod assembly and click into place.



7 Remove and discard both keys.



8 Remove plastic sleeve and band from new curtain.



9 Open new curtain and activate Timestrip®.

Timestrip® is a registered trademark of Timestrip UK Ltd.

TEAM Medical, a division of TEAM Technologies, brings together a portfolio of solutions for the healthcare industry with a variety of products ranging from PPE to specialized devices. Our TEAM of experts offers end-to-end solutions for manufacturing, supply chain and distribution needs as well as products used in healthcare settings ranging from the clinic to commercial applications.

Our Brands/Products:

- **Rapid Refresh Privacy Curtains**
- **Protexer PPE Apparel and disposable products**
 - Gowns, Scrubs, Shoe Covers, Bouffant Caps, Lab Coats, Sleeve Covers, Beard Covers, Stretcher Sheets
- **Protexer PPE Organizers**
 - Customizable Configurations, Glove Boxes, Enclosed Cabinets, PPE Dispensers
- **Protexer Sticky Mats**
- **BootieButler Shoe Covers**
- **DoseLogix**
 - Topi-CLICK
- **Pure Disinfectant**
- **ICP Medical Keyboards**

For more information contact:

TEAM Technologies

5949 Commerce Blvd
Morristown, TN 37814
toll-free: **1-423-587-2199**
sales@teamtech.com
www.teamtech.com

ICP Medical

13720 Rider Trail North
St. Louis, MO 63045
toll-free: **1-800-405-3044**
info.ICPMedical@teamtech.com
www.icpmedical.com

BootieButler

1616 Park 370 Ct.
St. Louis MO 63042
toll-free: **1-800-710-9863**
info.BootieButler@teamtech.com
www.bootiebutler.com

DoseLogix

300 Parkbrook Place, Suite 140
Woodstock, GA 30189
toll-free: **1-877-870-8448**
info.DoseLogix@teamtech.com
www.doselogix.com

www.teamtech.com

